

## Optional Setup with Atmospheric Inflation



### Inflate each Section of the Mattress Overlay to atmospheric Pressure:

**Step 1:** lay the overlay on to the foam overlay, attach each section using the push buttons.

**Step 2:** Open one of the inflation valves on each section (ensuring only one is open and the other is locked off)

**Step 3:** Inflate each section of the Overlay until the cells feel hard and firm.


**Step 4:** Remove the hand pump, but do not close the inflation valve.

**Step 5:** Allow air to escape, when you can no longer hear the air going out of the Mattress section close the valve.

**Step 6:** Place the cover provided over the system, the client can now use the mattress overlay.



**Open**

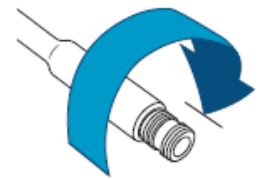

**Close**

**Manufacturer address:** Permobil Ltd, Unit 7 Polaris centre, 41 Brownfields, Welwyn Garden City, Hertfordshire, AL7 1AN.

**Email:** [info@permobil.co.uk](mailto:info@permobil.co.uk)

**Telephone:** 01484 722888