

Atmospheric Adjustment for ROHO® DRY FLOATATION® Cushions

1 Inflate the cushion.

Slide the Rubber nozzle of the hand pump over the inflation valve (figure 1). Pinch the nozzle and open the inflation valve (figure 2). Inflate until the cushion arches slightly.

2 Atmospheric setup.

Remove the hand pump. Do not close the inflation valve. Allow air to escape from the inflation valve.

Listen (or place your hand in front of the inflation valve) to confirm that deflation has stopped.

3 Finish setup.

Turn the inflation valve clockwise to close (figure 3).

Place the cushion on the surface, making sure the air cells face the individual.

Note: When possible, use the hook or loop fasteners to further secure the product to the surface.

Note: For the majority of individuals who use ROHO cushions,, atmospheric adjustment will work for set up. We always recommend a hand check to confirm there is enough air.

⚠ CHECK INFLATION! After atmospheric adjustment, confirm that there is **NO CONTACT** between the individual and the base of the cushioning device. If there is contact, add air.

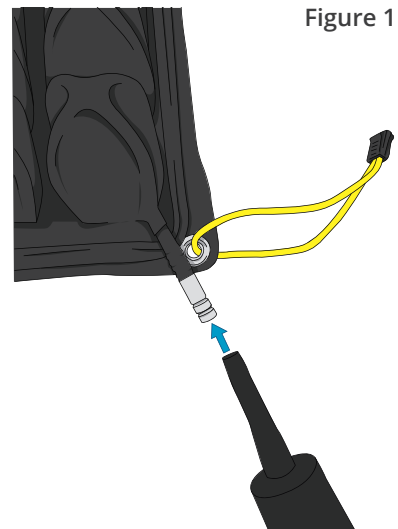
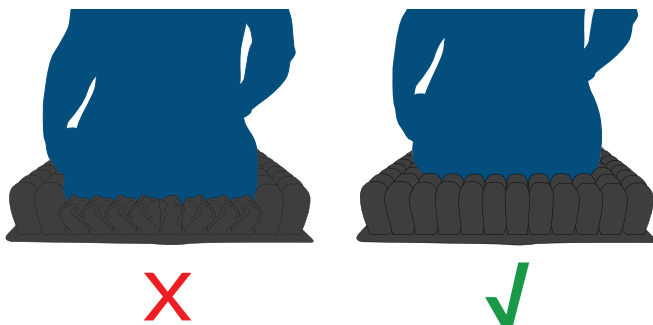


Figure 1

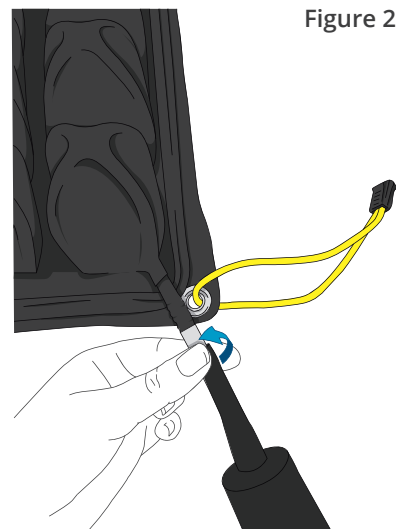


Figure 2

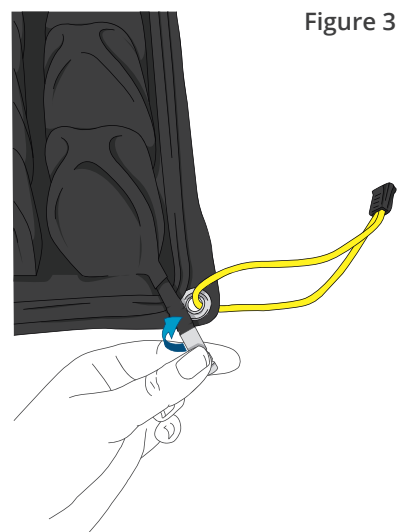



Figure 3

Contact Information:
 800-736-0925 Fax: 406-522-8563
 Customer Support: orders.comfort@permobil.com
 permobilus.com

 ROHO INC, LLC,
 100 North Florida Avenue, Belleville, Illinois 62221-5429 USA

ROHO INC, LLC is a part of Permobil Seating and Positioning.

Products may be covered by one or more U.S. and foreign patents and trademarks, including Permobil®, ROHO® and DRY FLOATATION®.