

The Series3 Auto: Quick Guide



BASIC SET-UP:

Plug in and switch on with the green switch on the side of the pump



Press the power button on the top of the pump. The pump will start up and the 8 pressure indicator lights will light up



When fully inflated, the 8 pressure indicator lights will flash continuously.



The auto weigh function will work once the user is placed on the mattress. Once complete, the 8 lights will stop flashing and the weight setting will set to the suggested ideal level for the user.



YOU DO NOT HAVE TO DO ANYTHING MORE THAN THIS.
Default set up is alternate mode.

EXTRA FUNCTIONS:

DECISIONS TO CHANGE THESE TO BE MADE BY COMPETENT MEMBERS OF STAFF.

To change the setting of the pump, unlock by pressing and holding the padlock for 3 seconds.

Comfort Settings – the weight scale is for comfort (soft to firm) and not essential to align with the person's actual weight. You can nurse a 100kg person on the 40kg setting. Press and hold the weigh button for 2 seconds.

Static Mode (Continuous low pressure) – stops the alternating function but retains pressure relieving properties and is an option if patients are unable to tolerate the alternating function.

Max firm – for short periods to provide a firm surface to move on or for personal care. The mattress will revert to normal after 20 minutes.

Seat inflate – this will increase mattress pressure to support the seated patient in the sacral area. This will need to be turned on and off manually.

RESOURCES

Manual



Troubleshooting Guide



Training Video

